




# Quick Guide: A/H7N9 Influenza Rapid Test

<b>Add Lysis Buffer Cap &amp; Invert Tube</b>	<b>Swirl Clinical Specimen in Lysis Solution</b>	<b>Apply Solution to Cassette 'S' Well</b>	<b>Read Results at 20 minutes</b>
---------------------------------------------------	------------------------------------------------------	------------------------------------------------	---------------------------------------


<i>Transfer <b>14 Drops</b></i>	<i>Transfer <b>4 Drops</b></i>
-------------------------------------	------------------------------------

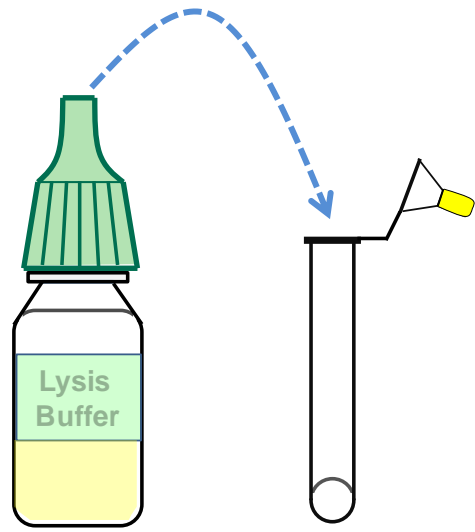

LYSIS TUBE

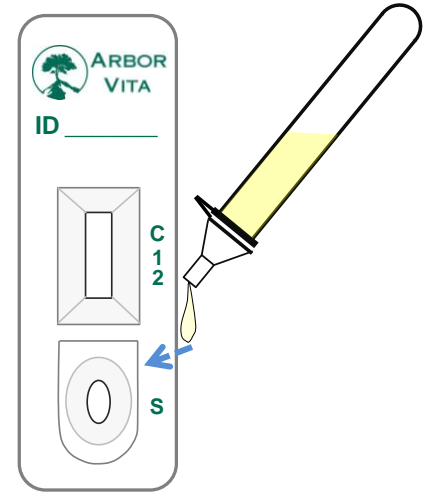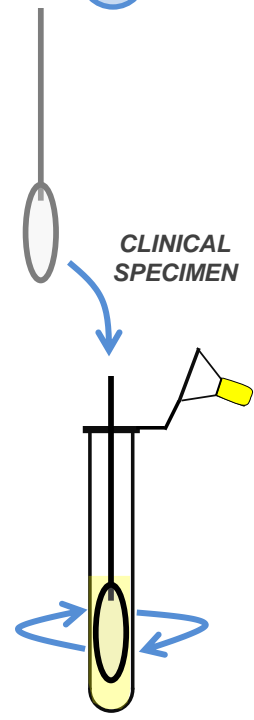

TEST CASSETTE

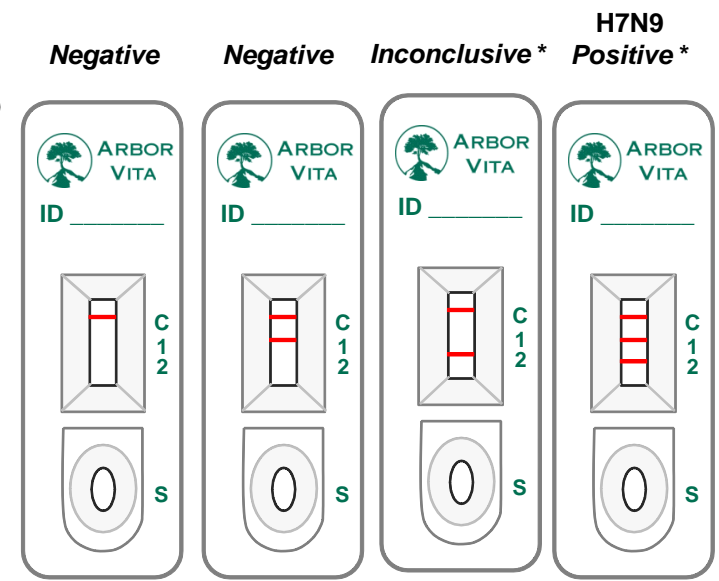

*\*FOR APPROPRIATE FOLLOW UP AND OTHER TEST OUTCOMES PLEASE REFER TO THE INSTRUCTIONS FOR USE*

<b>Solution Transfer &amp; Mixing</b>	<b>Sample Lysis</b>	<b>Sample Run</b>	<b>Result Interpretation</b>
-------------------------------------------	-------------------------	-----------------------	----------------------------------

<i><b>10 Seconds</b></i>	<i><b>20 Minutes</b></i>
--------------------------	--------------------------