
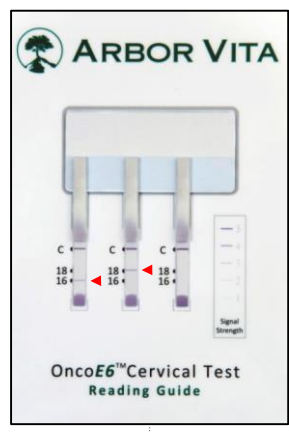
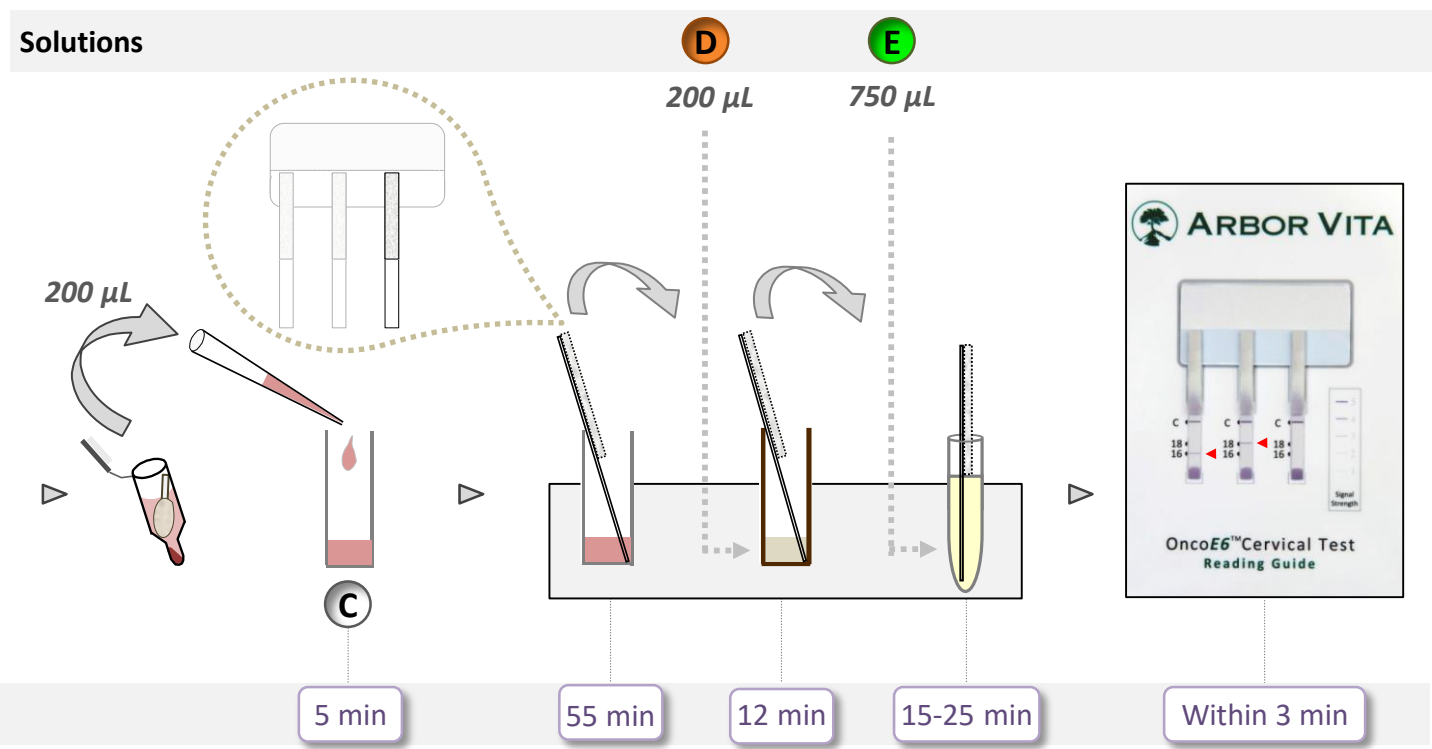


OncoE6™ Cervical Test

Quick Guide for Cervical Swab Specimen


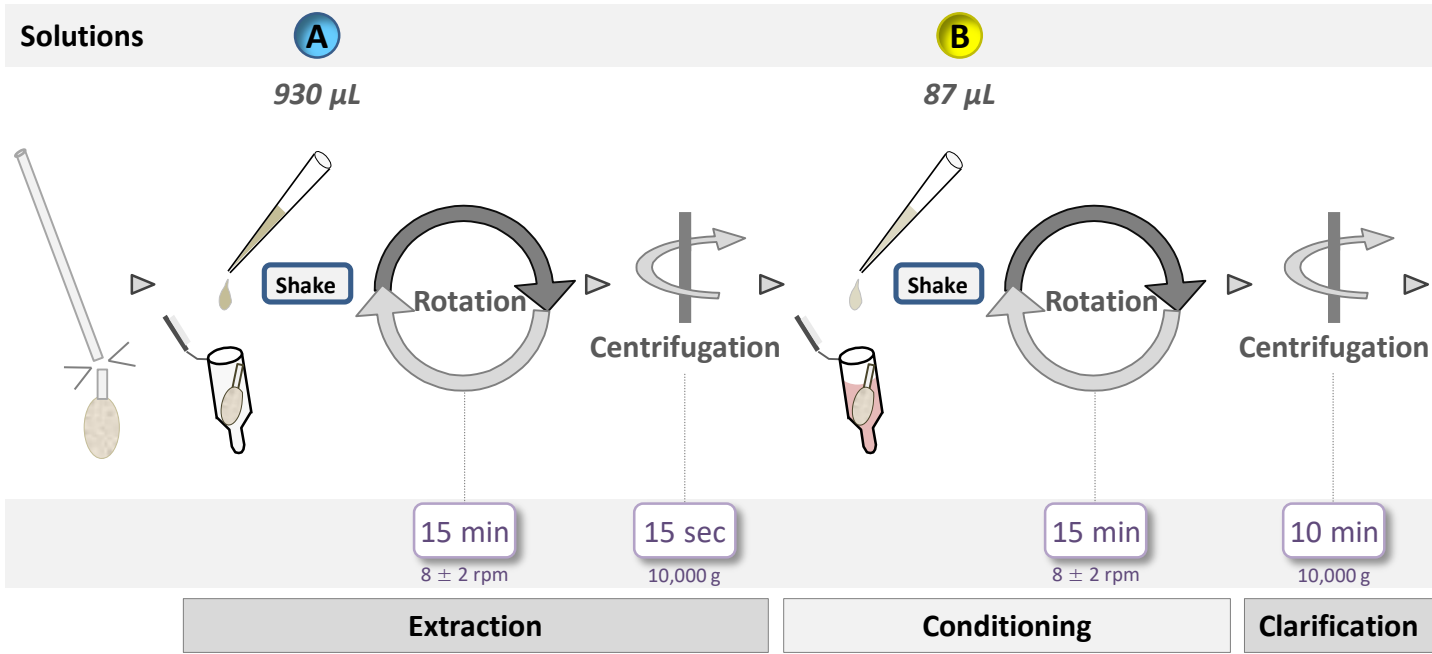


-Add 200 µL into Detector vial (C)
 -Swirl vial for 5-10 seconds
 -Incubate for 5 minutes
 -Swirl vial again for 5-10 seconds

Temperature (°C)	15.0 – 20.9	21.0 – 25.0	25.1 – 30.0
Development Time	25 min	20 min	15 min

OncoE6™ Cervical Test

Quick Guide for PreservCyt® Specimen

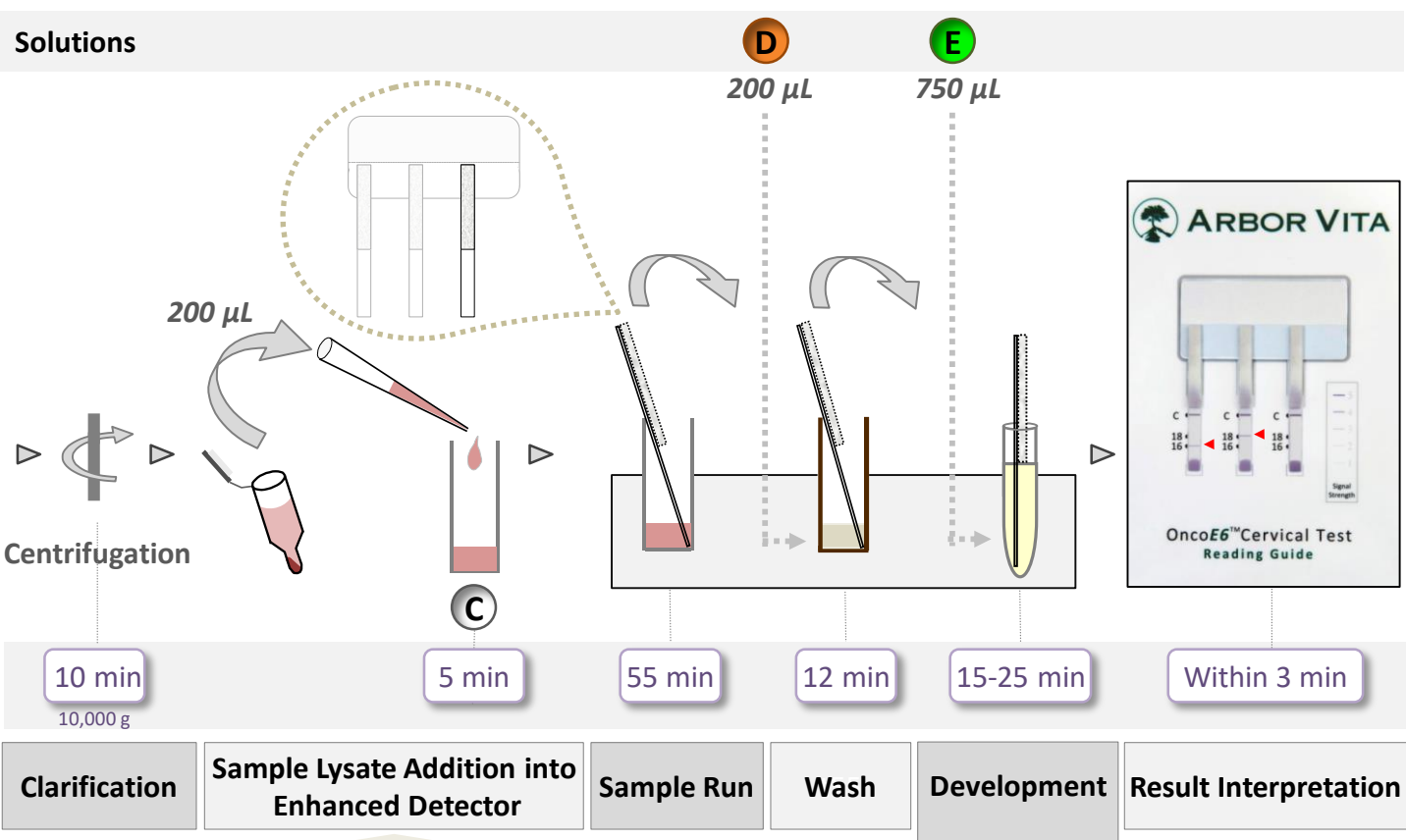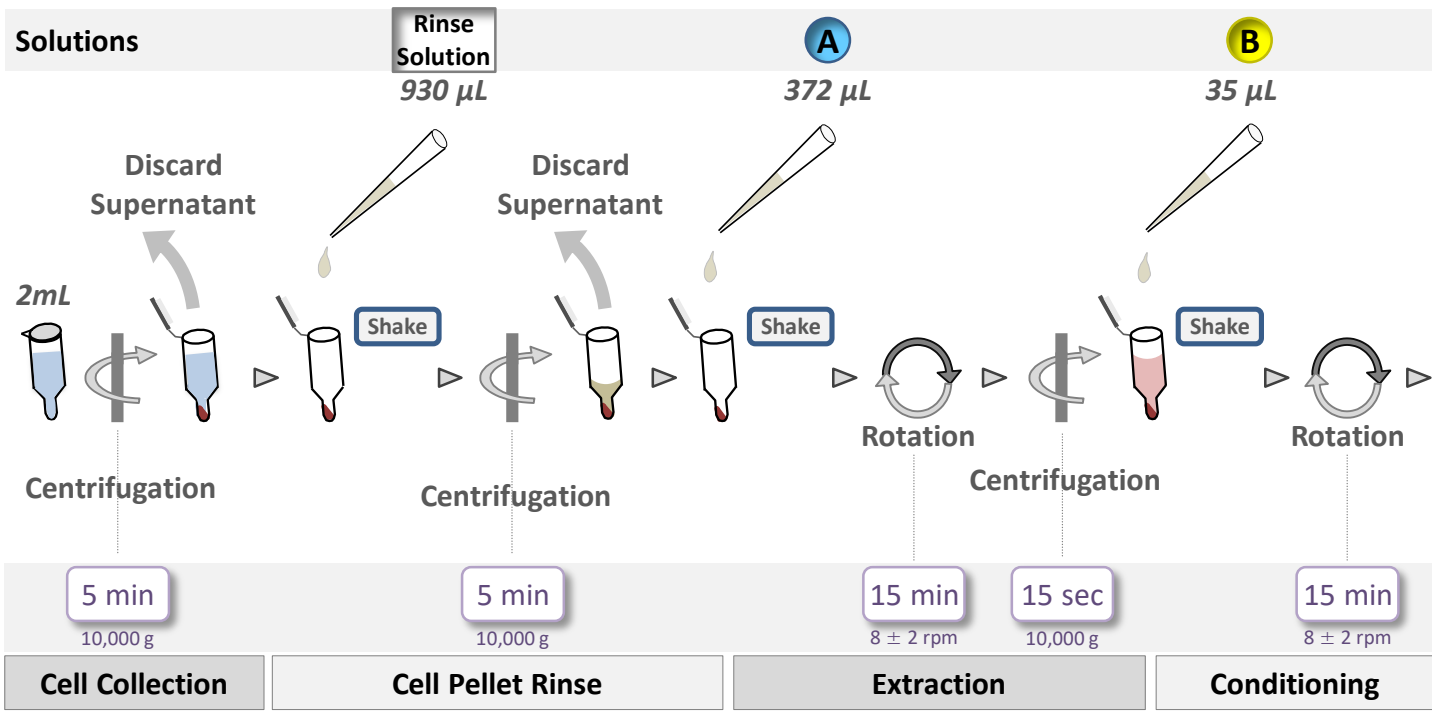

- Add 200 μL into Detector vial (C)
- Swirl vial for 5-10 seconds
- Incubate for 5 minutes
- Swirl vial again for 5-10 seconds

Temperature ($^{\circ}\text{C}$)	15.0 – 20.9	21.0 – 25.0	25.1 – 30.0
Development Time	25 min	20 min	15 min

